

Product range

Model	Art. no.	Standard sizes	Uni size
Foot wrap normal / long	J701LI	• 1 – 5	
Calf wrap normal / long available for left or right	J702LI	• 1 – 5	
Knee wrap normal	J703LI	• 1 – 5	
Thigh wrap normal / long available for left or right	J704LI	• 1 – 5	
Hand wrap normal	J705LI	• 1 – 4	
Arm wrap normal / long available for left or right	J706LI	• 1 – 5	
Hip band	J711		•
Midline band supplementary band for the upper extremities	J712	• 1 – 2	
Velcro fasteners	J713		•
Pads set with 2 thicknesses	J714	• 1 – 4	
Anti-slip band 12 x 115 cm	J715		•
Extender	J716		•

Size charts

Foot wrap

Art. no. J701LI	Circumference c Length ℓ	Size - Measurements in cm				
		I / XS	II / S	III / M	IV / L	V / XL
cA	over forefoot	22.0 – 24.0	25.0 – 27.0	28.0 – 30.0	31.0 – 34.0	35.0 – 40.0
	normal	16.5	18.0	19.5	21.0	23.0
	long	18.5	20.0	21.5	23.0	25.0

Calf wrap

Art. no. J702LI	Circumference c Length ℓ	Size - Measurements in cm				
		I / XS	II / S	III / M	IV / L	V / XL
cD'	largest point of calf	35.0 – 45.0	40.0 – 52.0	48.0 – 60.0	55.0 – 65.0	60.0 – 70.0
	ankle	20.0 – 27.0	25.0 – 32.0	30.0 – 38.0	37.0 – 44.0	45.0 – 52.0
	normal	37.0	37.0	42.0	42.0	44.0
	long	42.0	42.0	47.0	47.0	49.0

* below patella

Knee wrap

Art. no. J703LI	Circumference c	Size - Measurements in cm				
		I / XS	II / S	III / M	IV / L	V / XL
cF	10 cm above patella	40.0 – 55.0	50.0 – 65.0	60.0 – 75.0	70.0 – 85.0	85.0 – 100.0
	below patella	35.0 – 45.0	40.0 – 52.0	48.0 – 60.0	55.0 – 65.0	60.0 – 70.0

Thigh wrap

Art. no. J704LI	Circumference c Length ℓ	Size - Measurements in cm				
		I / XS	II / S	III / M	IV / L	V / XL
cG	largest point of thigh	50.0 – 65.0	60.0 – 75.0	70.0 – 80.0	75.0 – 90.0	85.0 – 100.0
	10 cm above patella	45.0 – 52.0	53.0 – 59.0	60.0 – 66.0	67.0 – 73.0	74.0 – 80.0
	normal	23.0	23.0	23.0	23.0	23.0
	long	29.0	29.0	29.0	29.0	29.0

Hand wrap

Art. no. J705LI	Circumference c	Size - Measurements in cm			
		I / XS	II / S	III / M	IV / L
cA	over knuckles	< 18.0	18.0 – 20.0	20.0 – 22.0	> 22.0

Arm wrap

Art. no. J706LI	Circumference c Length ℓ	Size - Measurements in cm				
		I / XS	II / S	III / M	IV / L	V / XL
cG	upper arm	25.0 – 28.0	29.0 – 33.0	34.0 – 39.0	40.0 – 45.0	46.0 – 50.0
	elbow	23.0 – 26.0	26.0 – 30.0	29.0 – 34.0	33.0 – 39.0	38.0 – 42.0
	wrist	14.0 – 16.0	17.0 – 19.0	20.0 – 22.0	23.0 – 25.0	24.0 – 27.0
	normal	42.0 – 47.0	42.0 – 47.0	42.0 – 47.0	42.0 – 47.0	42.0 – 47.0
	long	46.0 – 50.0	46.0 – 50.0	46.0 – 50.0	46.0 – 50.0	46.0 – 50.0

ADJUSTABLE COMPRESSION SYSTEM

Juzo® ACS light



high working pressure



durable



standard sizes

EN - 14030003/2/4 - W481 - 05/2017 - Subject to change, no liability for errors.



Juzo® ACS light

Efficient and effective adjustable compression systems

These short-stretch compression wraps can be employed as an efficient supplementary treatment during the decongestion phase. The Juzo ACS light products can be adjusted for the individual wearer by means of velcro fasteners. Thanks to the incorporated lining, they are easy to use and can be applied in next to no time. This ease of application increases patient compliance. Therapeutic compression pressures up to 40 mmHg are possible for arm and leg.


Features

- + high working and low resting pressure
- + compression levels up to 40 mmHg
- + can be worn day and night
- + adaptable whilst wearing for optimal compression
- + easy to apply – even with limited dexterity and mobility
- + patients can easily self-manage thanks to the built-in liner
- + can be worn with Juzo Compression Stockings, Sleeves or Gloves and also with Juzo SoftCompress Products
- + various accessories available

Material composition

85 % Polyamide
15 % Elastane

Colours

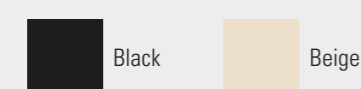

Upper extremities

Juzo ACS light Products for the upper extremities are compression tools with short-stretch characteristics, intended to reduce oedema in the arm and hand. Juzo ACS light Arm wraps can be combined with Juzo ACS light Hand wraps range. The wraps for arm are left and right specific – to enable a better fit and to make them easier to put on and take off. The Juzo ACS light Hand wraps are particularly suitable to treat oedema in the back of the hand.



Velcro fasteners

The products can be adjusted for the individual wearer by means of velcro fasteners.



Perfect hold

The segments can be applied in next to no time and fit perfectly thanks to the integrated liner. The sewn-in silicone border prevents the product from slipping.



Lower extremities

Juzo ACS light Products for the lower extremities are compression tools with short-stretch characteristics, intended to reduce oedema in the foot, lower leg, knee and thigh. Each part can work in combination with the other. The calf and thigh wraps are left and right specific – to enable a better fit and to make them easier to put on and take off. The foot bandage and knee bandage can be used for both left and right and the knee has a unique cut-out in the reverse for comfort.



Wearing comfort

A pressure pad which is loosely enclosed for the arm and hand segment offers additional comfort.



Anatomical fit

The compression segments are optimally adjusted to the different body parts. The segments for calf, thigh and arm are available for left and right.

Indications

- lymphoedema
- lipoedema
- post-traumatic oedema
- post-operative oedema

Contra indications

Absolute contra indications

- moderate to severe arterial disease (ankle-arm-index < 0,7)
- reduced sensitivity of the extremities
- untreated vascular infection
- uncontrolled (decompensated) congestive heart failure
- untreated or advanced skin disease
- swelling of the extremities of unknown cause
- advanced peripheral vascular disease
- untreated open wounds
- erysipelas

Relative contra indications

- peripheral neuropathy with diabetes mellitus
- congestive heart failure

Accessories



Juzo® SoftCompress

Special-purpose oedema products

Improves the treatment outcome

Juzo SoftCompress Products are employed as an effective treatment aid for compression bandaging in oedema treatment. Using the comfortable bandages and accessories can significantly improve the treatment outcome. The comfortable Juzo SoftCompress Bandages can be worn under Juzo ACS light Products, short-stretch bandages or medical compression stockings, and provide additional compression, cushioning and an ideal pressure distribution. The Juzo SoftCompress Products are especially skin-friendly and breathable. The material is washable up to 60 °C.

For further information regarding the Juzo SoftCompress product range please see the brochure "Special products for oedema therapy".

Juzo® Liner

Underliner for adjustable compression systems

Juzo Liner is an underliner for the ACS light Products. It preserves sensitive skin and simultaneously protects the wound dressing when donning or doffing adjustable compression systems.

The black stocking is delivered in pairs. Juzo Liner is available in two different sizes, both as calf and thigh stocking.

For further information as well as the size chart please see the brochure „Product range Phlebology · Lymphology · Scar therapy“.

