

OPTIVO[®] by OPED hand



Medium density dorsal strut for extension control and impact protection.



Wrist control strap to increase immobilisation and aid one handed donning.



Multi layers of fabrics and breathable materials to increase comfort.



Window for text indicating orientation of brace.



Contourable aluminium palmer strut for stability. Can be flipped to suit left or right.




Nylon loop for easy removal of strut to be adjusted to left or right.

Advantages at a glance.

- Excellent adjustment to degrees of swelling
- Low stock requirements thanks to the ability to use on the right or left side
- Optimally adjustable to the anatomy of the hand
- Multi layered, Skin-friendly padding material
- Super smooth, comfortable and light weight

Indications

- Carpal tunnel Syndrome
- Following removal of plaster cast dressings
- Wrist Sprains
- Tenosynovitis
- Osteo arthritis of the wrist
- Rheumatoid arthritis of the wrist
- Small distal radial fractures

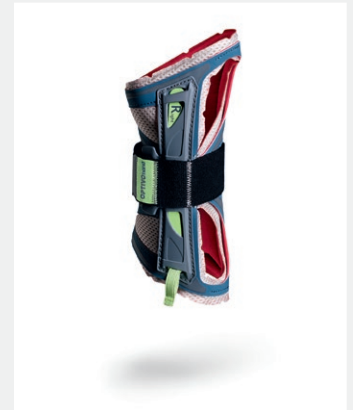

| Stock Code | Wrist Circumference Sizing | Side | Size |
|---------------|----------------------------|-----------|----------------|
| OPHA-01-1-S/M | Up to 17 cm | Universal | Small / Medium |
| OPHA-01-1-L | More than 17 cm | Universal | Large |

APPLICATIONS



Check that the brace is set up for the correct side.



If not, remove the aluminium strut, swing it around insert at the opposite end.



Slide brace on with thumb through the thumb window. Tension the wrist control strap firmly to secure the brace in place.



Then fasten the Velcro tabs as required.

ADDITIONAL PRODUCTS



SUPROhand



VACOhand RADIUS



OAPL Shoulder Immobiliser

*Prices do not include GST.

Orthopaedic Appliances Pty Ltd reserves the right to make any amendments and/or variations to this product or packaging without notice.